




Important Notice: Living cancer cells are required. These are very fragile once they are removed from the body. The biopsy specimen should be placed **immediately** into **cold** (just removed from the refrigerator) transport medium and held under refrigeration and/ or with cold packs until courier pick-up.

Specimen Transport Pack Instructions

Transport Pack Contents

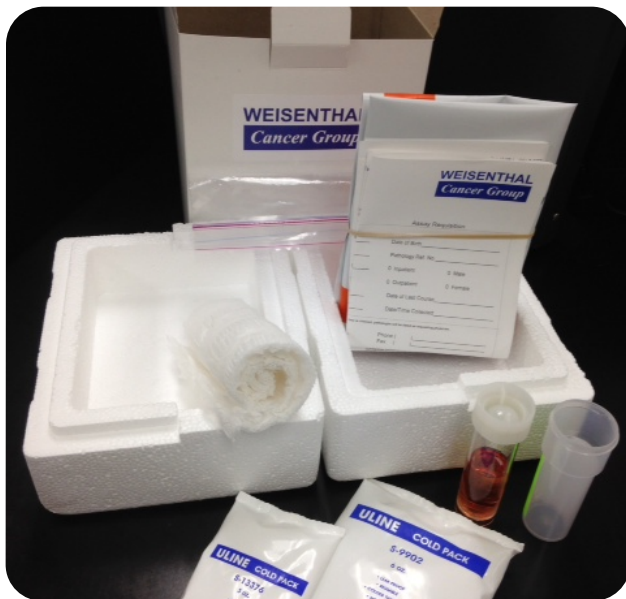

- Cardboard Outer Box
- Assay Requisition Form
- FedEx Clinical Pak
- FedEx Shipping Label
- Specimen Instructions
- Plastic Zip-Lock Bag
- Styrofoam Case
- Absorbent Wadding
- Outer Specimen Vial
- Inner Specimen Vial
- Large Cold Pack
- Small Cold Pack

Packing Instructions

1. Drop the inner vial (with specimen and media) into the outer plastic vial.
2. Wrap the nested vials in absorbent wadding.
3. Place the wadding and vials in the small plastic bag and zip securely.
4. Place the large cold pack in the styrofoam case.
5. Place the plastic bag containing the specimen on top of the large cold pack.
6. Place the small cold pack next to the specimen.
7. Slide the styrofoam case into the cardboard outer box.
8. Place the completed requisition form inside the cardboard box (but not inside the styrofoam case) or else just inside the FedEx Clinical Pak.
9. Place the cardboard box inside the FedEx Clinical Pak, seal the pouch, and affix the label.
10. Affix the FedEx shipping label to the outside of Clinical Pak.

